


GH3+ Integrated Weigh Scale

Operating Instructions

Warning! If the scale is required to comply with the Non-automatic weighing instruments directive (EU Directive 2014/31/EU), a Class III scale must be used.

Resetting the Unit [Tare]

1. Press any key on the hand control to activate the GH3+. When activated, a display on the hand control is switched on and the Guldmann logo "G" appears.
2. Select "Menu" using the function key located immediately below the display.
3. Then select until the "Scale" menu appears in the display.
4. Then select "Start".
5. Park the hand control on the lifting hanger. (If the helper pulls the hand control during weighing, this will affect the result of the weighing process).
6. When the lifting hanger with the lifting sling is at rest, select "Tare" to reset the GH3+ scale module. The scale module has now been reset and weighing can commence.

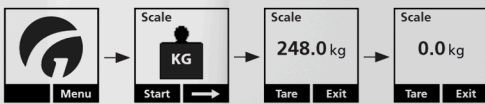

Weighing

1. Always reset the GH3+ scale module before weighing takes place.
2. Position the lifting sling on the user and attach it to the lifting hanger.
3. Lift the user with care.
4. Park the hand control on the lifting hanger.
5. Select "Menu" using the function key located immediately below the display and then select until the "Scale" menu appears in the display.
6. Then select "Start".
7. When the sling and the user are at rest and hanging freely, the current weight can be read on the display.
8. Select "Exit" to return to the main menu.

